
- Ⓓ Montageanleitung
- ⒼⒷ Installation instructions
- Ⓕ Notice de montage
- ⒹⒻ Montage-instructie
- Ⓘ Istruzioni di montaggio
- Ⓔ Instrucciones de montaje
- ⒶⒻ Instrukcja montażu

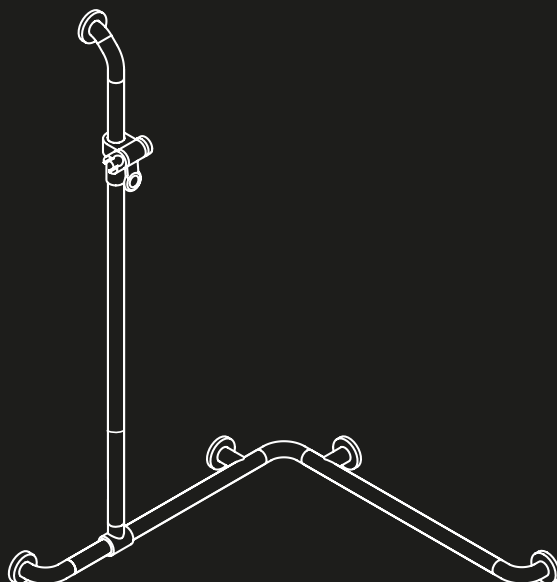Web: www.unidomo.de

Telefon: 04621- 30 60 89 0

Mail: info@unidomo.com

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VIESSMANN

 **Vaillant**

WOLF

 **JUNKERS**  **BOSCH**

 **remeha**

 **DAIKIN**

ROTEX
a member of DAIKIN group

Individuelle Beratung

Kostenloser Versand

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Komplettpakete

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Markenhersteller

D	HEWI Support	0049 5691 82-300 (D), 00 800 4394 4394 (A, CH Freecall Festnetz)
GB	HEWI support	0044 1634 258200 (UK), 0049 5691 82-0 (International)
F	Support HEWI	0033 472 83 09 09 (F), 00 800 4394 4394 (BE, LU, CH appel gratuit, poste fixe)
NL	HEWI support	00 800 4394 4394 (BE, LU, NL gratis, vaste lijn)
I	HEWI supporto	0049 5691 82 0
E	HEWI soporte	0049 5691 82 0
PL	HEWI wsparcie	0048 022 330 0052

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Safety advice

- Use the HEWI product only in the way illustrated.
- Improper installation/use can cause accidents and injuries.
- The installer must pass on the installation and use instructions to the user and they must be kept for the whole life of the product.

Mounting

- On acceptance the product must be checked for damage; any defects must be reported immediately.
- The product must be stored in a dry place and free from pressure. The wall on which the seat is to be installed must be flat, firm and capable of taking the load. Only use fixing materials approved by the wall anchor manufacturer for the intended purpose. In case of doubt, contact HEWI's sanitary products customer service.
- Ensure that the wall is suitable for mounting the product.
- Mounting only by persons familiar with mounting of the product and appropriate qualifications for their activity.

Utilisation

- The form of the upper curve makes it difficult to fix ropes or cords at the top.
- Bottom fixing point should be installed on a height of 85 cm maximum.
- Do not use the hinged seat for any purpose other than that for which it is intended, e.g. do not use it as gymnastics equipment or as a climbing aid or step.

Maintenance

Maintenance of the shower and bath grab rail or of the system as a whole should be performed on a 6-monthly basis and should encompass the following key points:

1. Inspection of fixing

The shower and bath grab rail should be mounted firmly at all fixing points and should fit tightly to the surface to which it is fixed. If necessary, the rose caps should be removed and the screws used to fix the rail in place tightened up. Any defects identified during inspection should only be remedied by persons qualified to do so.

2. Inspection of surface finish of grab rail.

The surface finish of the rail should have no fractures

or other damage that could cause personal injury. There should be no gaps at the joints between the straight elements and the bend elements.

Care and cleaning instructions

HEWI products are easy care and easy to clean. Frequently, occasional wiping with a damp cloth is sufficient. If using cleaning products the following should be noted:

- Only use cleaning products with a pH value between 6 and 8.
- Cleaning products that contain detergents such as phosphates, soaps and tensides (surfactants) as their active substances can be used.
- Please note and follow the cleaning product manufacturers' instructions (concentration, contact time).
- Do not use alkali, acidic or chlorine-based cleaning products or cleaning products with scouring agent additives.
- Do not use any scouring cleaning equipment, e.g. brushes.
- Do not apply the cleaner directly to the surface, but always pour it on the cleaning cloth instead.

Disclaimer

HEWI does not accept any liability whatsoever for personal injuries or damage to property that is or has been caused due to incorrect assembly, installation or use, especially due to disregard of the listed warnings and safety instructions.

Disposal

Proper disposal is subject to the applicable national statutory regulations.

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HEWI Heinrich Wilke GmbH