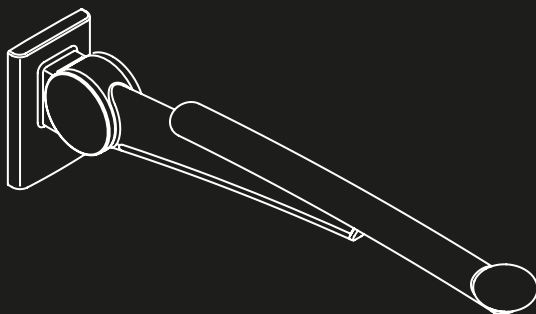
- Ⓓ Montage- und Gebrauchsanleitung
- ⒼⒷ Installation and Operating Instructions
- Ⓕ Notice de montage et mode d'emploi
- ⒶⒻ Installatie-en gebruikshandleiding
- Ⓘ Istruzioni di montaggio e per l'uso
- Ⓔ Instrucciones de instalación y funcionamiento
- ⒶⒻ Instrukcja montażu i obsługi

Web: www.unidomo.de

Telefon: 04621- 30 60 89 0

Mail: info@unidomo.com

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VIESSMANN

 **Vaillant**

WOLF

 **JUNKERS**  **BOSCH**

 **remeha**

 **DAIKIN**

ROTEX
a member of DAIKIN group

Individuelle Beratung

Kostenloser Versand

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Komplettpakete

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Markenhersteller

D	HEWI Support	0049 5691 82-300 (D), 00 800 4394 4394 (A, CH Freecall Festnetz)
GB	HEWI support	0044 1634 258200 (UK), 0049 5691 82-0 (International)
F	Support HEWI	0033 472 83 09 09 (F), 00 800 4394 4394 (BE, LU, CH appel gratuit, poste fixe)
NL	HEWI support	00 800 4394 4394 (BE, LU, NL gratis, vaste lijn)
I	HEWI supporto	0049 5691 82 0
E	HEWI soporte	0049 5691 82 0
PL	HEWI wsparcie	0048 022 330 0052

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Proper use in accordance with Directive 93/42/EWG

HEWI hinged support rails have been designed for use in bathrooms and toilets. They provide grip and support, and assist the independent standing and sitting of persons with and without physical impairment.

Warning

- The HEWI hinged support rails are designed for a maximum static working load of 100 kg vertically.
- A certain amount of vertical sag under load is normal. In case of damage, e.g. exceeding of the working load, the hinged seat must no longer be used.
- Do not use the hinged seat for any purpose other than that for which it is intended, e.g. do not use it as gymnastics equipment or as a climbing aid or step.
- When the seat is folded down ensure that there is no part of the body under the product.
- Improper installation/use can cause accidents and injuries.
- Mounting only by persons familiar with mounting of the product and appropriate qualifications for their activity.
- The wall on which the produkt is to be installed must be flat, firm and capable of taking the load. Only use fixing materials approved by the wall anchor manufacturer for the intended purpose. In case of doubt, contact HEWI's support tel. 0044 1634 258200 (UK), 0049 5691 82-0 (International).
- After installing the seat, perform a functional test.
- On acceptance the product must be checked for damage; any defects must be reported immediately. The product must be stored in a dry place and free from pressure, e.g. no other items stacked on top of it.
- The installer must pass on the installation and use instructions to the user and they must be kept for the whole life of the product.
- The toilet roll holder is not suitable as a carrying handle for hinged support rails.
- The maximum static working load of hinged support rails does not increase the because of the floor support.

Maintenance

The HEWI hinged support rail is not maintenance-free! It must be checked monthly by qualified personnel to ensure that it is in perfect working order. In case of defects (e.g. braking force too small) the relevant steps given in the installation and use instructions must be taken.

Care and cleaning instructions

HEWI products are easy care and easy to clean.

Frequently, occasional wiping with a damp cloth is sufficient. If using cleaning products the following should be noted:

- Only use cleaning products with a pH value between 6 and 8.
- Cleaning products that contain detergents such as phosphates, soaps and tensides (surfactants) as their active substances can be used.
- Please note and follow the cleaning product manufacturers' instructions (concentration, contact time).
- Do not use alkali, acidic or chlorine-based cleaning products or cleaning products with scouring agent additives.
- Do not use any scouring cleaning equipment, e.g. brushes.
- Do not apply the cleaner directly to the surface, but always pour it on the cleaning cloth instead.

Disclaimer

HEWI does not accept any liability whatsoever for personal injuries or damage to property that is or has been caused due to incorrect assembly, installation or use, especially due to disregard of the listed warnings and safety instructions.

Disposal

Proper disposal is subject to the applicable national statutory regulations.

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(GB) Mounting only by persons familiar with mounting of the product and appropriate qualifications for their activity.

BM10.3 - concrete (synthetic dowel)
BM11.3 - aircrete
BM12.3 - lightweight walls with rear lining
BM13.3 - lightweight walls with steel plates
BM17.3 - concrete
BM18.3 - solid bricks, perforated bricks, lightweight concrete
Please follow dowel producers instructions to set drill holes.

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Use spanner (2) to pull setting element (1) out of bearing (3) as far as it will go. **Caution – risk of injury!** Once setting element has been disengaged, the force of the spring acts directly on spanner!

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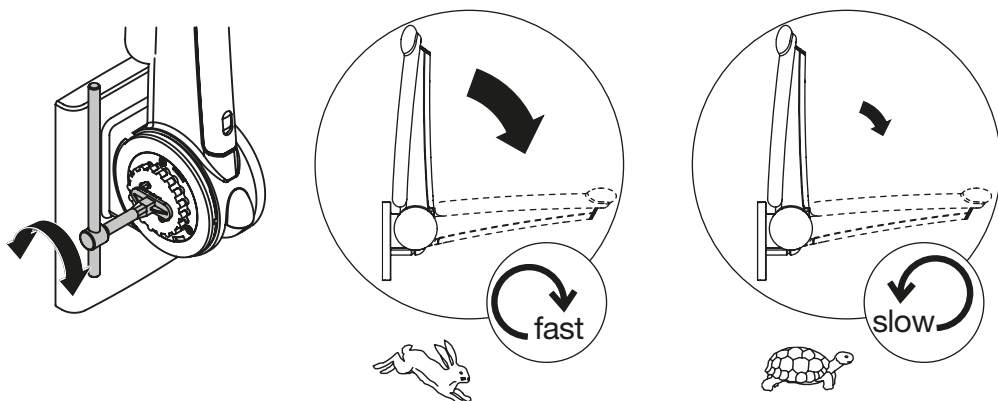
Turn setting element with socket spanner until the required spring tension has been reached.

To increase spring tension: turn spanner in anticlockwise direction.

To reduce spring tension: turn spanner in clockwise direction.

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HEWI Heinrich Wilke GmbH