Pflege

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Maintenance

The Robert-Koch-Institute recommends that: "Lotion dispensers must be thoroughly cleaned and sterilised whenever cartridges are changed. The system should be flushed several times with hot water, in order to remove deposits. Only completely emptied containers are to be refilled. The same requirements apply for the use of skin care lotions".

For cleaning we explicit recommend warm water and soap. Abrasive or chlorine-containing cleaners are unsuitable. The dosage of fluids containing salt or chlorine can lead to corrosion at the pump even though the pump is made of high-quality stainless steel. Also, the installation of the dispenser in areas with chlorine content in the air (close to swimming pools, saunas etc.) can cause corrosion of the stainless steel parts. In order to ensure perfect continued functionality of the pump, a regular cleaning is necessary in the dishwasher, in the autoclave, or by hand.

Do not dismantle the unit. Please only use original equipment manufacturer spare part. Only use the dispenser for the dosage of suitable liquids such as soap, lotions or disinfectants. **Caution!** Using inappropriate liquids can cause the dispenser to malfunction and splash liquid into your eyes.

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



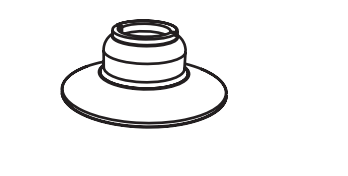
Adapter for bottles

The included adapter for bottles ensures the optimal contact pressure of the bottle and prevents the volatilization of alcohol contained in the disinfectant.

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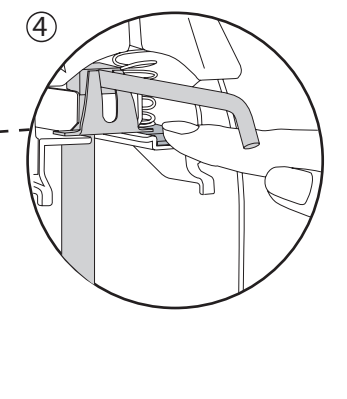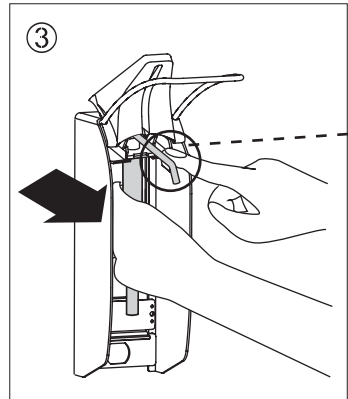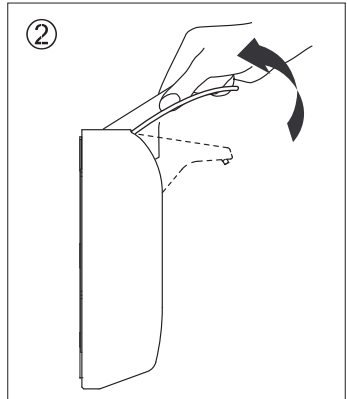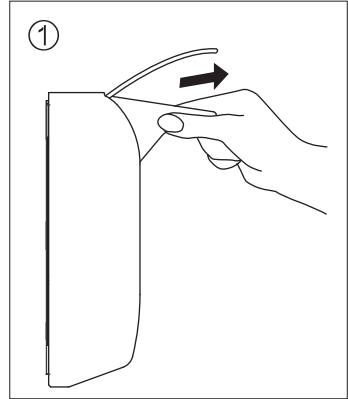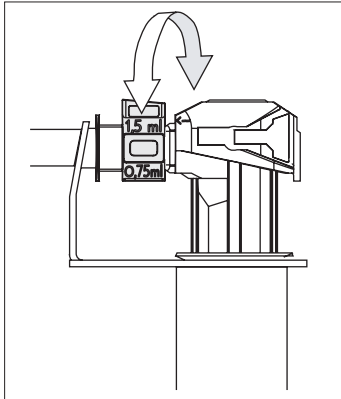
Pump removal

Slightly pull outlet face shield, then lift it in order to press the pump lock as shown in picture 3+4. Carefully pull out pump.

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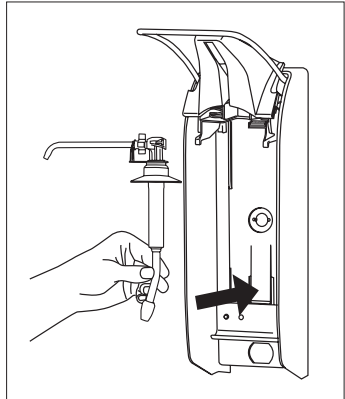
Dosage

The dispenser is adjusted to the maximum amount of approx. 1.5 ml/stroke. Dosage adjustable at approx. 0,7 ml/1,0 ml/ 1,2 ml/1,5 ml (in dependence on Viskosity of the product).

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Cleaning/Sterilizing

Press fixing catch and remove dispenser from the wall by pushing it upwards. Aluminium or stainless steel dispenser (including the stainless steel pump) are autoclavable at 134 °C, 3 Bar.

Wall plate and bottle are not autoclavable!

Attention: After removing the pump from the autoclave, dispenser and pump have to cool down for at least one hour before re-use.

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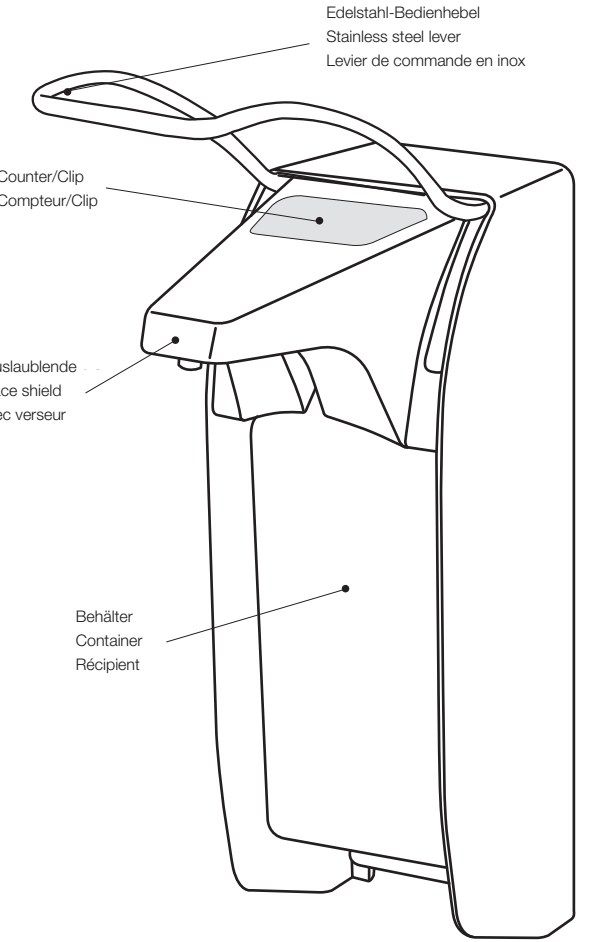
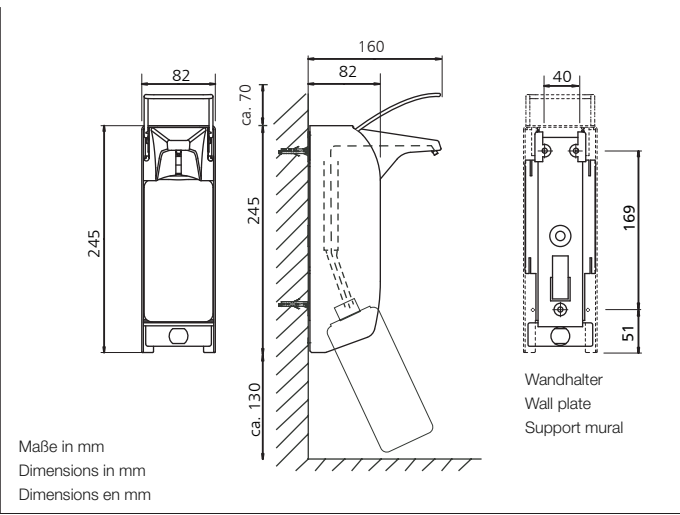
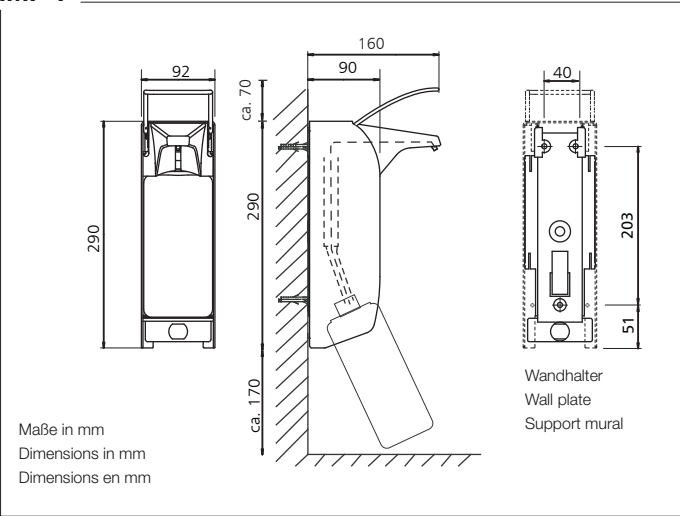
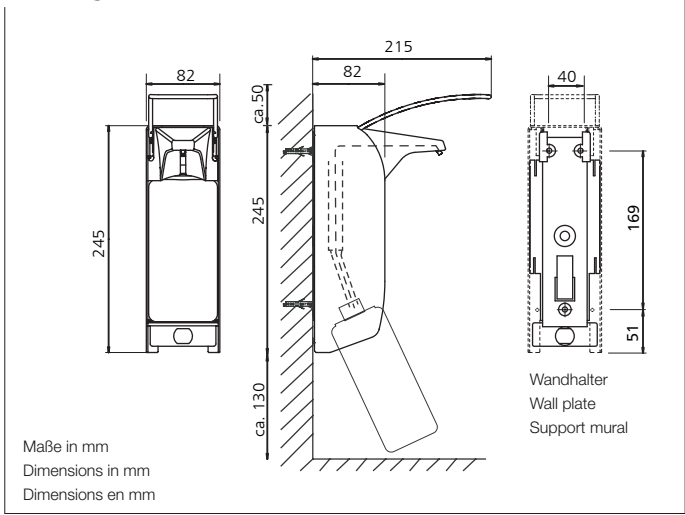
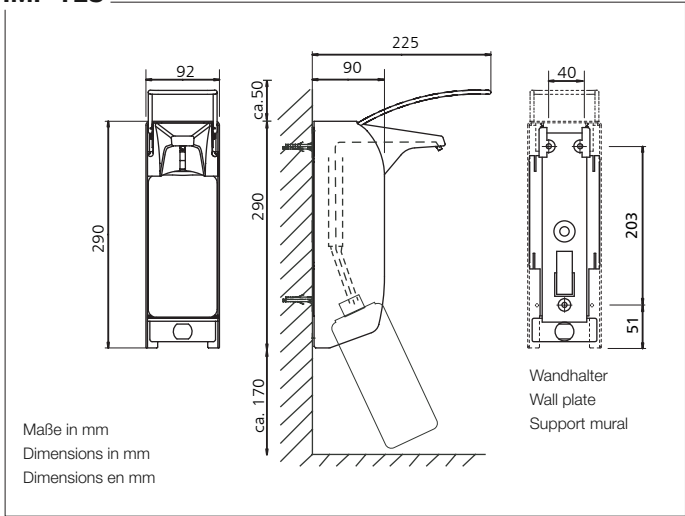
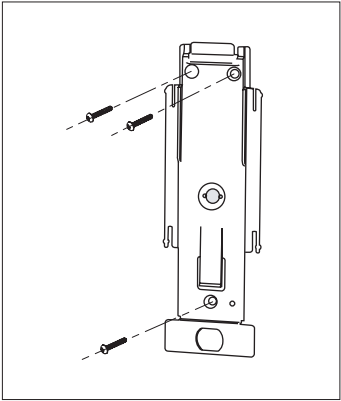
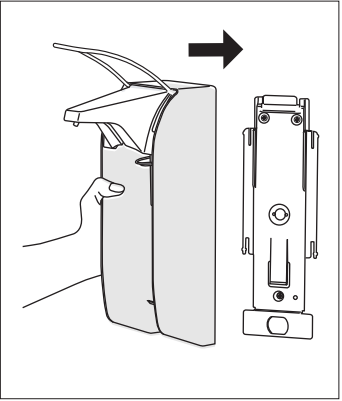
Note before mounting

1. Install the dispenser above or close to the sink in order to prevent wet hand from dripping on the floor.
2. Keep at least 50-70 mm distance above the dispenser to the next object; another 100 mm for operating the lever. The distance below the dispenser needs to be at least 130 mm for IMP E/ELS or 170 mm for IMP T/TLS.
3. Optimal mounting height: 130-135 cm from floor to top of dispenser.

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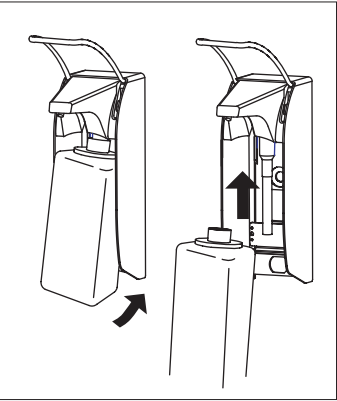
Mounting

Remove the container. Slightly press the fixing catch and pull down the wall plate. Fix wall plate with enclosed plugs and screws. Firmly push the dispenser from above onto the wall plate until it clicks into place. Please check adequacy of the wall first. Already installed standard ingo-man dispenser wall plates are compatible to the HEWI dispenser.

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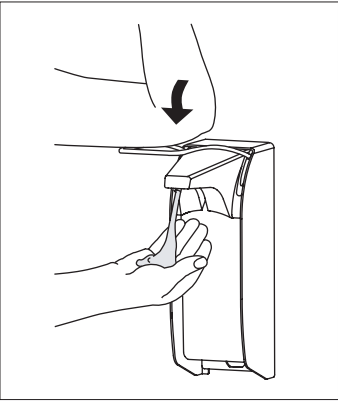
Container exchange

Insert the refilled container from below.

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Operation

For initial working, the unit must be operated several times until it delivers full portions. Hold your hand under the outlet nozzle.

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