- Ⓓ **Montageanleitung**
- ⒼⒷ **Mounting instructions**
- Ⓕ **Notice de montage**
- ⓃⓁ **Montage-instructie**
- Ⓘ **Istruzioni di montaggio**
- Ⓔ **Instrucciones de montaje**
- Ⓟ **Instrukcja montażu**

Web: www.unidomo.de

Telefon: 04621- 30 60 89 0

Mail: info@unidomo.com

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VIESSMANN

 **Vaillant**

WOLF

 **JUNKERS**  **BOSCH**

 **remeha**

 **DAIKIN**

ROTEX
a member of DAIKIN group

Individuelle Beratung

Kostenloser Versand

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Komplettpakete

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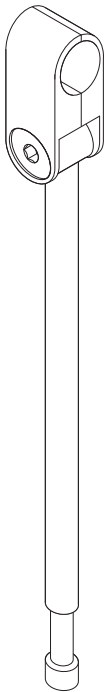
Markenhersteller

950.50.02...

	a	b	x	Abb. Nr.
950.50.026XA	680 mm	563±20 mm	665 mm	Abb. 1
950.50.020XA	760 mm	643±20 mm	745 mm	Abb. 1
950.50.025XA	850 mm	733±20 mm	835 mm	Abb. 1
950.50.021XA	760 mm	607±20 mm	743,5 mm	Abb. 2
950.50.022XA	760 mm	607±20 mm	743,5 mm	Abb. 2
950.50.023XA	850 mm	697±20 mm	833,5 mm	Abb. 2
950.50.024XA	850 mm	697±20 mm	833,5 mm	Abb. 2

Abb. 1



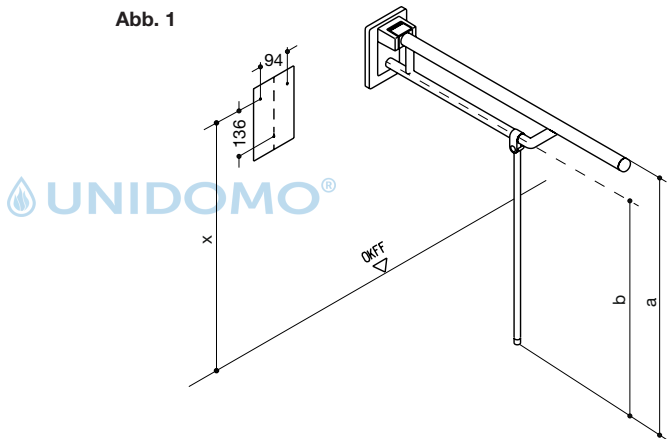
Abb. 2

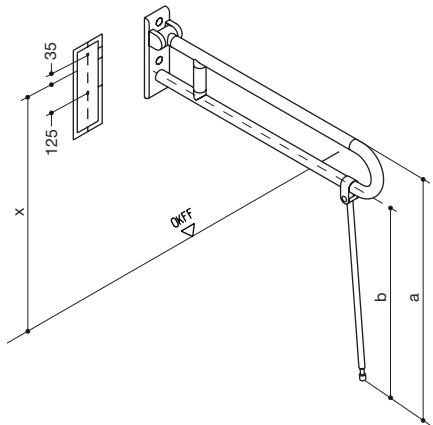D

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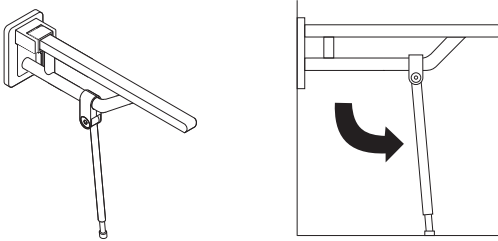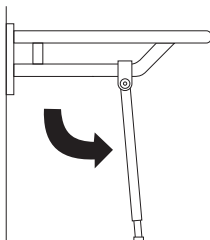
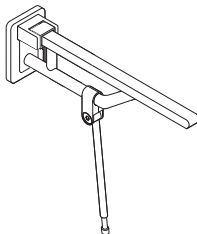
Purpose according to Directive 93/42/EEC

The HEWI floor support is an accessory for the HEWI support rail and serves to take the strain off the wall fixing of the HEWI support rail.

Mounting information

Mounting only by persons familiar with mounting of the product and appropriate qualifications for their activity. Ensure that the floor covering is suitable for the permanent use of the floor support and able to withstand the load.

Warning and hazard notices

- Always unfold the floor support fully.
- The floor support must stand firmly on the floor.
- Improper mounting or use can lead to accidents involving serious injuries.
- The maximum static load of the support rail is **not** increased by using the floor support.
- In the event of damage, the floor support rail may only continue to be used following inspection by a specialist company responsible for maintenance.

Care and cleaning instructions

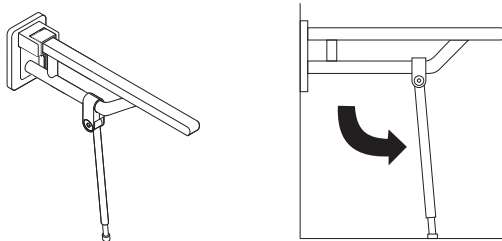
HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. When using detergents, please observe the following:

- Only use detergents with a pH of between 6 and 8.
- Detergents containing active substances such as phosphates, soap and tensides may be used.
- Follow the detergent manufacturer's instructions (concentration, contact time).
- Detergents containing acids, alkalis, bleach or scouring agents may not be used.
- Do not use scouring cleaning equipment, such as brushes

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of improper mounting or maintenance, particularly if the safety instructions and other points to note listed are not or have not been observed in full. HEWI accepts no liability whatsoever for damage occurring from incorrect cleaning.

**For further information, please contact HEWI Technical Support,
tel. +49 5691 82-300.**

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HEWI Heinrich Wilke GmbH